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BIO-ALOE-VERA-GEL

ÄTHERISCHES BIO-ORANGENÖL

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CARBOXYMETHYL-BETA-GLUCAN = MUKO-ADHESIVE ACTION

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EN Instructions for use

Eureka Care - Otosan® Nasal Spray Baby & Kids

SLIGHTLY HYPERTONIC SOLUTION (1%) OF SEA WATER FROM BRITTANY WITH ORGANIC ALOE VERA GEL, ORGANIC ORANGE ESSENTIAL OIL, COPPER GLUCONATE, AND CARBOXYMETHYL BETA-GLUCAN.

USE. The Eureka Care - Otosan® Nasal Spray Baby & Kids medical device is a nasal decongestant that acts locally through osmosis helping to reduce nasal congestion, and indirectly, inflammation. Eureka Care - Otosan® Nasal Spray Baby & Kids is a sea water hypertonic solution (1%) enriched with Carboxymethyl Beta-Glucan, Copper Gluconate, Organic Aloe Vera, and Orange Essential Oil. The multi-action of Eureka Care - Otosan® Nasal Spray Baby & Kids has local effects cleaning the nasal cavities and freeing the nose from excess mucus, thus helping to reduce nasal congestion. Thanks to its delicate formula free of vasoconstrictors, this product is particularly suitable for children. Free from propellant gas.

INDICATIONS. Eureka Care - Otosan® Nasal Spray Baby & Kids is recommended as a first-line adjunctive therapy for inflammatory processes such as different types of rhinitis (viral, bacterial, or allergic), sinusitis, and dryness of the nasal mucosa.

DOSAGE AND TARGET POPULATION. This medical device is intended for use in children. However, it must be used under adult supervision. Consult your physician before using it in children aged 1-3 years. Recommended close 1 spray actuation per nostril 2-4 times daily, for 10-15 days.

DESCRIPTION. Contact between the hypertonic solution and the mucous membranes generates a secretion of water, recovered by osmosis, helping to dilute built-up thick mucus and facilitating its fluidification. The reduction of mucus viscosity improves mucociliary clearance and helps to remove viruses, bacteria, pollen, as well as micro and macro particles through the expulsion of mucus. The product components help to expel excess mucus and have protective, and indirectly, softening and soothing effects on the irritated mucosa. Thanks to the removal of excess secretions and to the treatment of the mucous membranes with natural substances that facilitate the creation of a hostile environment for bacterial or microbial contamination, Eureka Care - Otosan® Nasal Spray Baby & Kids provides relief and prevents relapses.

TOLERABILITY AND SIDE EFFECTS. The absence of vasoconstrictors and the gentle action of its ingredients ensure that Eureka Care - Otosan® Nasal Spray Baby & Kids is normally well tolerated, and its frequent use (not exceeding 15 days) does not cause tolerance or create a risk of «rhinitis medicamentosa». In rare cases, it can cause anosmia - due to the presence of copper-based components (Cu(II)).

In rare cases, it can cause epistaxis.

In rare cases, when there is a hypersensitivity to its components, it can cause a local allergic reaction. A slight tingling sensation may be experienced inside the nostril right after application. Should you experience side effects other than those described herewith, please contact the manufacturer immediately and consult your physician.

INSTRUCTIONS FOR USE. Blow your nose before using the spray. Spray the solution in each nostril, keeping your head upright. After spraying, slightly press the nostril with your index finger to ensure that the solution is distributed on the entire nasal mucosa. A slight and temporary tingling sensation after application is to be considered a normal effect of the product. After use, wash the nozzle in warm water and dry it with a tissue.

WARNINGS AND PRECAUTIONS.

- Store the medical device in a dry place, away from direct sunlight
- Do not use in case of known hypersensitivity to its components
- Use the medical device at room temperature (20°C - 25°C)
- Do not expose the bottle to a heat source
- Do not use in case of lesions of the nasal mucosa
- Keep out of children's reach
- Do not use in children under 1 year of age
- Consult your physician before using it in children aged 1 to 3 years
- Avoid contact with eyes
- Do not ingest
- Do not apply to damaged skin
- Do not apply in the ear canal
- Do not use in ways and for purposes other than those expressly specified by the manufacturer in these instructions for use
- Do not use after the expiry date
- Do not use in case of hypersensitivity to crustaceans and/or molluscs
- In the event of unwanted reactions or adverse effects, discontinue treatment immediately, consult your physician and notify the manufacturer
- The bottle is strictly for personal use; it might carry infections
- The components of Eureka Care - Otosan® Nasal Spray Baby & Kids have no known contraindications when used during pregnancy or breastfeeding; in these cases, consult your physician before using the product

A MULTI-ACTION PRODUCT THAT GENTLY CLEANSSES THE NASAL CAVITIES WITH A QUICK DECONGESTANT EFFECT.

SEA WATER (CONCENTRATE) = CLEANSING AND SOOTHING ACTION

The sea water used in the formula comes from the sea near Brittany. Its composition is similar to a saline solution and is particularly rich in mineral nutrients and trace elements. In addition to cleaning the nasal cavities and washing away dust and pollen, hypertonic sea water favours mucociliary clearance and fluidifies excess mucus, thus facilitating its expulsion.

ORGANIC ALOE VERA GEL

ORGANIC ORANGE ESSENTIAL OIL

COPPER GLUCONATE = PROTECTIVE ACTION

CARBOXYMETHYL BETA-GLUCAN = MUCO-ADHESIVE ACTION

It is a polysaccharide derived from Beta-Glucan and obtained by extracting it from the cell walls of yeasts and fungi.

DISPOSAL. The medical device must be disposed of in compliance with the waste management regulation of your local management authority. Do not dispose of the product or its packaging into the environment.