Mr. Aerosol Neb

Gebruikshandleiding • Manuel d'utilisation
Gebrauchsanweisung • Users manual

NL • FR • DE • EN

Fig. 1

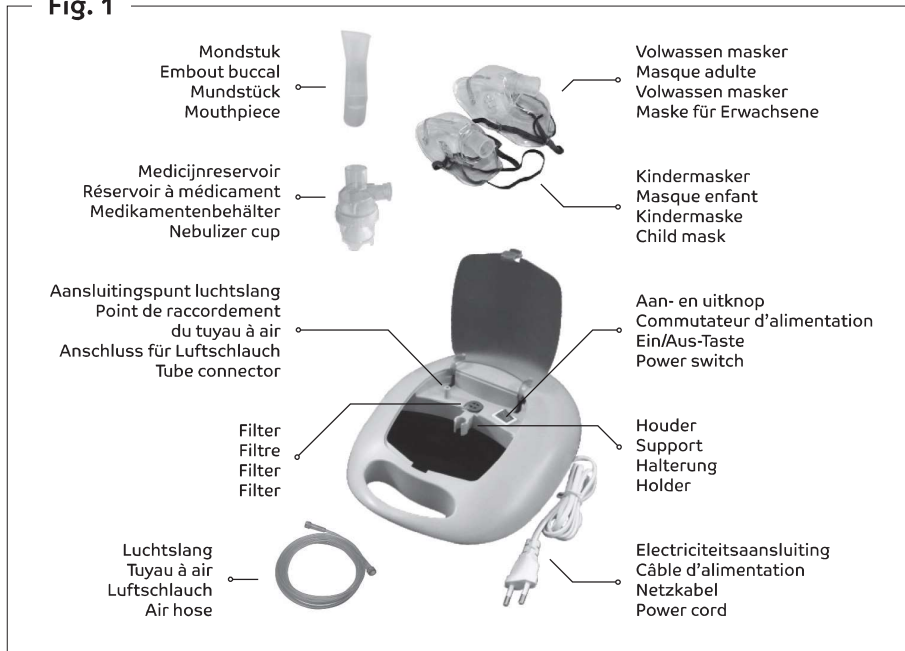
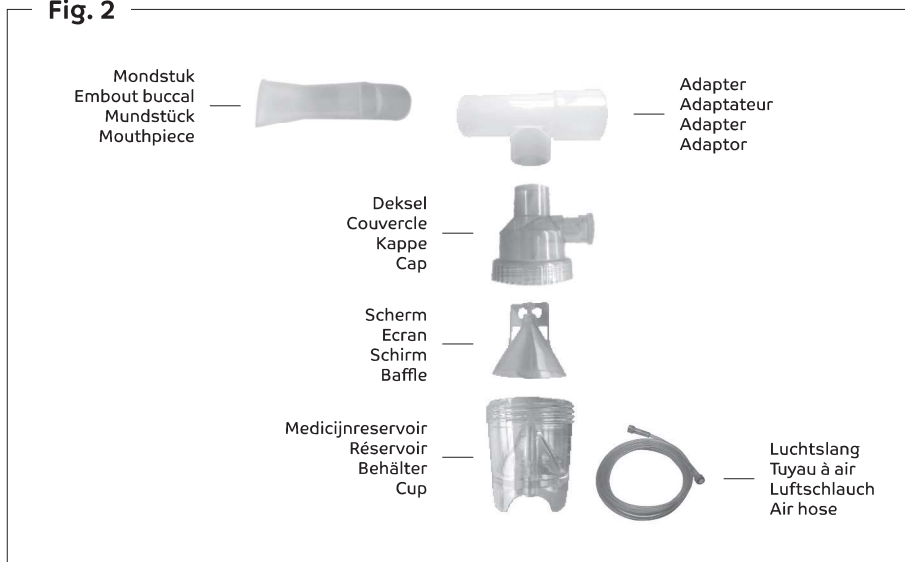
Fig. 2

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PLEASE READ THE INSTRUCTIONS CAREFULLY BEFORE OPERATING THE PRODUCT.

1. WHAT IS THE MR. AEROSOL NEB?

The Mr. Aerosol Neb is a medical device designed to deliver medication prescribed by a physician to the airways of the lungs. The nebulizer and air compressor combined provides an aerosol mist for inhalation by the patient for medical treatment only.

2. HOW DOES THE NEBULIZER WORK?

This unit is compact, simple to operate and convenient to carry. It uses a non-oil lubricating, single-cylinder piston pump to create air pressure. Air flows through the air filter into the nebulizer's air compressor unit, out the jet outlet, and then through the nebulizer.

3. CHECKLIST FOR UNPACKING

Before using this product, make sure you have the following items:

- Aerosol Nebulizer
- 1 medicine cup
- 1 aerosol tubing
- 1 mouth piece
- 2 masks
- 5 air filters

4. COMPONENTS

The standard nebulizer components are diagramed in figure 1 (page 2).

5. HOW DO I INSERT MEDICATION INTO THE NEBULIZER?

See figure 1 (page 2).

To insert medication into the Nebulizer, do the following:

1. Open the cap.
2. Fill the cup with medication as prescribed by your physician.
Be sure that the medication does not exceed the liquid site line of 6ml (6cc).
3. Close the cap.
4. Connect the mouthpiece.

NOTE: The mouthpiece is to be used for oral inhalation only.

6. HOW DO I OPERATE THE NEBULIZER?

1. Plug in the air compressor, but do not connect the Nebulizer.
2. Make sure that the air filter is attached to the air compressor.
3. Turn on the power switch.
4. After the air compressor runs properly for 1-2 minutes, connect the jet outlet of the air compressor to the inlet of the nebulizer by the air hose.
5. Begin treatment according to your physician's instructions.
6. Inhale and exhale naturally. Be sure to stop when the medication is completed.
7. When you have completed use, turn the air compressor unit off before you disconnect the air hose.

WARNING: Do not use near flammable objects or locations near explosive materials.

7. WHAT DO I DO AFTER I'VE USED THE NEBULIZER?

To keep your nebulizer working properly after each use, do the following:

1. After you have disconnected the air hose, disconnect the mouthpiece.
2. Open the cap and empty the cup. Wash the Nebulizer with running water or soak in warm water for 15 minutes. For a more thorough, hygienic cleaning, add some vinegar in the water.
NOTE: Do not boil the nebulizer. Doing so could damage it.
3. Dry the Nebulizer completely before storing it.

8. HOW DO I REPLACE THE AIR FILTER?

Generally, the air filter should be replaced every three months or after 300 uses.

To replace the air filter:

1. Remove the filter cover.
2. Replace the filter.
3. Re-attach the filter cover.

NOTE: Whenever the filter becomes wet, allow it to dry completely before using.

9. WHAT MAINTENANCE FOR STORAGE DOES THE NEBULIZER REQUIRE?

1. Always handle your nebulizer with care.
2. Disconnect the power plug from the wall outlet when not in use.
3. Keep your Nebulizer out of children's reach.
4. If you store your nebulizer, do not bend the air hose.
5. If you store your Nebulizer, try to keep in the following environmental ranges:
 - Temperature: -40°C - 55 °C
 - Relative humidity: < 95%
 - Atmospheric pressure: 0.05 Mpa - 0.106MPa
6. If possible, store your Nebulizer in a well ventilated room.

10. SPECIFICATIONS

Electric requirements	AC 220V/50Hz/60Hz
Working compressor pressure	0.7 bar
Compressor free air flow	> 8L/min
Sound level	≤ 60 dB(A)
Nebulizer net weight	1.5 kg
Compressor size	272 (L) x 215 (W) x 105 (H) mm
Electric safety	UL547 Class B
Maximum time running intermittently and stopping separately	30 min

11. TROUBLESHOOTING GUIDE

SYMPTON	PROBABLE CASUE	SOLUTION
Too much noise.	No filter in the unit.	Put a filter in the unit.
The nebulizer does not run.	Main compressor switch is off.	Turn the unit on.
	Nebulizer was not cleaned after its previous use.	Clean the unit.
	Air hose is bent.	Remove any bends or kinks in the air hose.
	The filter is clogged.	Replace the filter.
	No medication is left.	Add the appropriate amount of medication prescribed by your physician to the cup.
	The cap is not attached properly.	Re-attach the cap.
Water drops have formed on the air hose.	The medication was over-filled	Add the appropriate amount of medication to cup, connect the air hose to the compressor and turn the unit on.
	The nebulizer was washed but not dried.	Open and close the nebulizer repeatedly with your finger untill the drops disappear.

NOTE : For further questions, contact Eureka Pharma at info@eurekapharma.be

eurekapharma.be

 Jinghao Medical Technology Co, LTD
Made in PRC